



Select the next action from the toolbar on the right.

Date 2022/4/8 15:22:42 Case ID 00006-002

Client 00006 10086

Name file,test

Technician

Type notes here...

THE FIRST STEP :
Please click below "Open in explorer" selection at first when EXO software opened.

Project status

Material configuration (local): Default




file_test-preview

Scan > CAD

Open in explorer

Job definition Multi-die mode

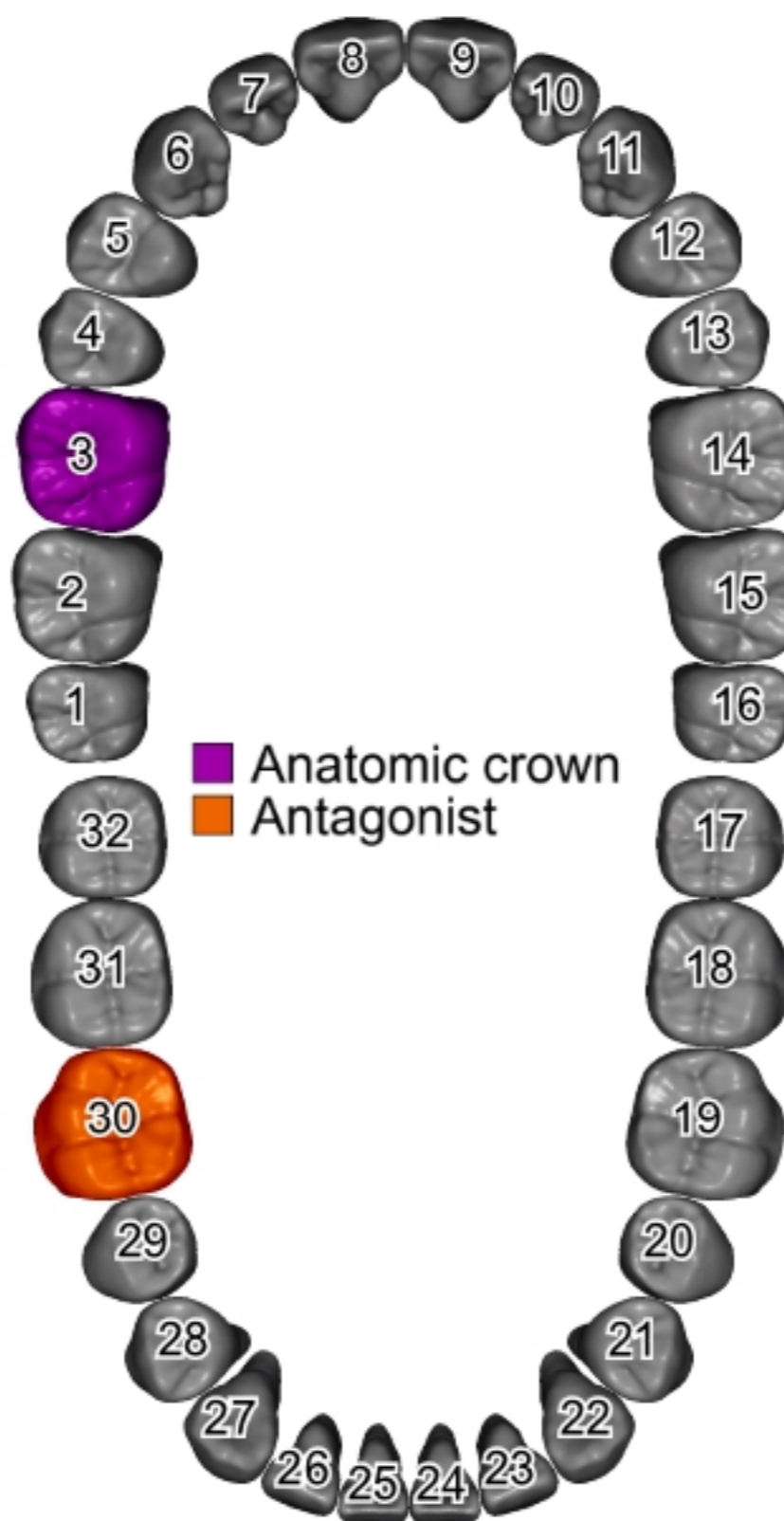

Material shades

A1

Scan mode

Two stone models in occlu

- Scan
- Design
- Manufactur
- Model Creator
- dentalshare
- Copy

New Load Save

Select the next action from the toolbar on the right

Date 2022/4/8 15:22:42 Case ID 0

Client 00006 10086

Name file,test

Technician

Type notes here...

Project status

Material configuration (local): Default



file_test-preview

Scan CAD Open in explorer

Material shades

A1

Scan mode

Two stone models in occlu

file_test

文件 主页 共享 查看

固定到快速访问 复制 粘贴 剪贴板 剪切 复制路径 粘贴快捷方式 移动到 复制到 删除 重命名 新建项目 轻松访问 新建文件夹 属性 打开 全部选择 全部取消 反向选择 选择

此电脑 > 本地磁盘 (D:) > CAD-Data > file_test

名称	修改日期	类型	大小
cad file_test	2022/4/8 15:23	DENTALPROJEC...	12 KB
file_test-lowerjaw	2021/11/19 17:16	STL 文件	12,580 KB
file_test-occlusion	2021/11/19 17:16	STL 文件	8,029 KB
file_test-preview	2022/4/8 15:23	PNG 文件	267 KB
file_test-upperjaw	2021/11/19 17:16	STL 文件	14,803 KB

本地磁盘 (C:) 本地磁盘 (D:)

5 个项目

THE SECOND STEP :

Please compress all above files into one folder in ZIP format.